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## User's Manual RES-Q-VAC – Manual suction pump

EN

This appliance conforms with the Directive 93/42/CEE "Medical Devices"

Guarantee of Quality system for the production and the final control of the products certified by the notifying body TÜV SÜD Product Service GmbH

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EN

|                                 |   |
|---------------------------------|---|
| 1 - GENERAL INFORMATION         | 6 |
| 2 - WARNINGS                    | 6 |
| 3 - DESCRIPTION OF PRODUCT      | 6 |
| 4 - OPERATING INSTRUCTIONS      | 7 |
| 5 - MAINTENANCE AND CLEANING    | 8 |
| 6 - ACCESSORIES AND SPARE PARTS | 8 |

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## 1 GENERAL INFORMATION

### 1.1 AIM AND CONTENTS

The aim of this manual is to supply all the information necessary so that the client, will not only attain adequate use of the appliance, he will also be capable of using the instrument in the most autonomous and secure way possible. This includes information regarding technical aspects, functioning, maintenance, spare parts and safety.

### 1.2 CONSERVATION OF THE USER MANUAL

The user manual must be kept with the product, inside the specially provided container and above all, away from any substances or liquids which could compromise perfect legibility.

### 1.3 SYMBOLS USED

| SYMBOL                                                                                 | MEANING                                                                     |
|----------------------------------------------------------------------------------------|-----------------------------------------------------------------------------|
|       | General or specific warning                                                 |
|       | See instructions for use                                                    |
|  / SN | Lot number / Serial number                                                  |
|       | Product code                                                                |
|       | The product is compliant with the specifications of the Directive 93/42/CEE |

### 1.4 SERVICING REQUESTS

For any information regarding the use, maintenance and installation, please contact the Spencer Customer Care Service on 0039 0521 541111 - fax 0039 0521 541222 - e-mail: info@spencer.it or write to Spencer Italia S.r.l. - Via provinciale 12 - 43038 Sala Baganza (PR) - Italy.

### 1.5 DEMOLITION

When devices and their accessories are no longer suitable for use, they can be disposed of as normal municipal solid waste if they have not been contaminated by special agents. Otherwise, follow the regulations in force regarding disposal

### 1.6 LABELLING

The serial number as indicated below can be found on each appliance and must not be removed or covered. In order to facilitate assistance please indicate or communicate the lot number (LOT) or serial number (SN) on the label.

## 2 WARNINGS

### 2.1 GENERAL WARNINGS

- Before carrying out any kind of operation on the appliance, the operator must carefully read the enclosed instructions, paying particular attention to the correct safety precautions and to the procedures to be followed for installation and for correct use.
- In the case of any doubts as to the correct interpretation of the instructions, please contact Spencer Italia S.r.l., for any necessary clarifications.
- Check the appliance before and after every use. Carry out the prescribed maintenance in order to keep the appliance in good condition and to guarantee correct functioning and a long life.
- In the case of any abnormalities or damage to the appliance, which could jeopardize the functioning and the safety, the appliance must be immediately removed from service.
- Do not alter or modify in any way the appliance; any such interference could cause malfunctions and injury to the patient and/or rescuer.
- The appliance must not in any way be tampered with. In such cases all responsibility will be denied for any malfunctions or injuries caused by the appliance itself. Those who modify or have modified, prepare or have prepared medical appliances in such a way that they no longer serve the purpose for which they were intended, or no longer supply the intended service, must satisfy the valid conditions for the introduction onto the market.
- Ensure that all the necessary precautions are taken in order to avoid the hazards that can arise as the result of contact with blood or body fluids.

- Handle with care.

### 2.2 SPECIFIC WARNINGS

- The device is only for use by trained staff.
- Do not leave the device exposed to atmospheric agents for a long time.
- Do not use to suck inflammable substances.
- Some accessories could not be compatible with the tubes supplied. It is essential to check the compatibility of each accessory before use.
- Use of the device may require special precautions to be taken to avoid the risks of contagious diseases both during use, cleaning and connection of the tubes and jar.
- After having dismantled and re assembled the device, the user must carry out the test procedures indicated in paragraph 4.3.
- If during rescue operations the device presents any dysfunction, it must be replaced with a similar type of device so that the suction procedure can be continued.
- A spare jar must always be available so as to guarantee the continuity of suction.
- During use of the device, the patient must always have a trained operator at his side.
- Do not use the device for thoracic drainage.

### 2.3 CONTRAINDICATIONS AND SIDE EFFECTS

There are no contra-indications or side effects, if the device is used as described in this user manual.

### 3.1 INTENDED USE

RES-Q-VAC is a suction device designed to collect non inflammable substances of only medical origin. RES-Q-VAC is an easy to use portable suction device particularly indicated for emergency situations; it does not require any power supply and it can be operated using the hands, knees or feet. The device is used to free the patient's airways.

All of its components are easy to remove and wash: RES-Q-VAC can be completely autoclaved. The accurate connections and the robust structure guarantee a long life span. The intervention on the device on behalf of the patient is not to be considered.

3.2 MAIN COMPONENTS

- The principal components of the device are:
- 1. Shell
  - 2. Pedal
  - 3. Collection jar
  - 4. Short tube
  - 5. Long tube
  - 6. Filter
- SHELL: encloses the whole device; manufactured in polycarbonate.  
PEDAL: permits the correct functioning of the device; it can be used either by pressure of the hand, foot or knee. Manufactured in stainless steel sheets coated with epoxy powder paint.  
COLLECTION JAR: permits suction and collection of substances: manufactured in polycarbonate. Jar vase and the suction device and between the jar and the suction tube (not supplied with the device).  
OVERFLOW VALVE: this valve will automatically close when the jar is full of suction liquid so as to avoid the liquids entering the internal parts of the pumps; it is manufactured plastic material.  
FILTER: avoids any type of bacterial or viral contamination for the rescuer during use. It must be substituted after each use of the device. Manufactured in polytetrafluoroethylene (PTFE).

3.3 TECHNICAL DATA

| DIMENSIONS               |          |
|--------------------------|----------|
| Width                    | 225 mm   |
| Height                   | 150 mm   |
| Depth                    | 165 mm   |
| Weight                   | 1,4 kg   |
| Maximum vacuum           | 300 mmHg |
| Average flow             | 22 l/min |
| Volume of collection jar | 400 ml   |
| Suction hose length      | 1300 ml  |

3.4 ENVIRONMENTAL CONDITIONS

|                                           |                                           |
|-------------------------------------------|-------------------------------------------|
| <b>During use:</b>                        | <b>During storage:</b>                    |
| Temperature from 5 to +35 °C              | Temperature from -40 to +70 °C            |
| Atmospheric pressure from 530 to 1100 hPa | Atmospheric pressure from 530 to 1100 hPa |
| Humidity from 30 to 75%                   | Humidity from 10 to 100%                  |

4 OPERATING INSTRUCTIONS

4.1 TRANSPORT AND STORAGE

- Before transporting the appliance, make sure that it is correctly packed ensuring also that there are no risks of shocks, bumps or falls during the transport itself.  
Keep the original packaging for use in case of any further transport. Damage to the appliance caused during transport and handling is not covered by the guarantee. Repairs or replacement of the damaged parts are the responsibility of the client. If the device is not to be used for a long period, in addition to the advice already given, precautions must be taken regarding where the device is stocked and for how long:
- Store the unit in a dust-proof place
  - Make sure the device is not subject to bump or vibrations.
  - Protect the device from humidity and maintain a stable temperature.
  - Make sure the device does not come into contact with corrosive substances.

4.2 PREPARATION

- On receipt of the device check that all parts are present.  
The device must be checked before each use to ensure that it is functioning correctly and ready for use.
1. Place the device on a flat and perfectly horizontal surface (ex. shelf, trolley, table).
  2. Insert the short tube on the suction outlet and into the lid on the vase in correspondence with the red float.  
The side of the protection filter with "IN" indicated on it must be attached towards the connection of the suction vase.  
**Connecting the tube the wrong way could cause un repairable damage to the device.**
  3. Insert the long tube on the connector on the lid of the vase.
  4. Action the pedal and close the end of the tube with your thumb to check that the device is sucking correctly.  
The device can be checked if the suction tube is immersed in a container of water.  
**Check that the overflow valve is closed. If the overflow valve does not shut off correctly, it could cause damage to the device.**

4.3 FUNCTIONING

- After completing the essential preparation routine, the device is ready for use.  
To limit frothing, unscrew the jar's lid and fill the jar 1/3 with water (this will allow easier cleaning operations and a rapid reaching of vacuum too), then screw the lid on the jar ensuring a proper sealing.  
Start suction by continually pressing and releasing of the pedal until suction is completed. The pedal can be pushed using the foot, hand or knee. The suction capacity is adjusted by the frequency of compression of the pedal.  
Be careful to monitor that the overflow valve remains closed.  
Keep a spare collection jar at hand ready for substitution if the vase in use is full.  
**When use terminates**  
When use of the device has terminated, the tubes must be disconnected (long and short ones) and the device must be accurately cleaned (paragraph 5) before placing the device back in the correct storage condition.

4.4 TROUBLESHOOTING

| PROBLEM                  | CAUSE                                                                          | REMEDY                      |
|--------------------------|--------------------------------------------------------------------------------|-----------------------------|
| The float has not closed | The float could have come un attached during the process of washing of the lid | Fit the float back in place |

IT  
EN  
DE  
FR

|                   |                                             |                                                                                                                                                                                |
|-------------------|---------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| No suction occurs | Mucus overflows - Filter blocked up         | 1. Substitute the filter.<br>2. Check the O ring is present and that it is in good condition.                                                                                  |
|                   | Tube connection is not correct              | Connect the long and the short tube correctly                                                                                                                                  |
|                   | Obstruction of tubes                        | Clean or substitute the tubes                                                                                                                                                  |
|                   | Tubes cut                                   | Substitute tubes                                                                                                                                                               |
|                   | Incorrect position of long tube (crinkling) | Check that the position of the tubes does not cause crinks                                                                                                                     |
|                   | The collection jar does not create vacuum   | 1. Check and/or screw on the lid. If the problem is not resolved contact the assistance centre.<br>2. Check the O-ring on the lid is present and that it is in good condition. |
|                   | When using the pedal, not vacuum is created | Contact the assistance centre                                                                                                                                                  |

## 6 MAINTENANCE AND CLEANING

### 5.1 CLEANING

To clean the plastic housing of the device wear disposable latex gloves and clean with denaturated alcohol or hypochlorite solutions. Washing and / or cleaning the autoclavable jar as to be carried out as follows:

- Wear protection gloves and apron (glasses and face mask if necessary) to avoid contact with contaminating substances;
- Disconnect the jar from the device
- Disconnect all tubes from the jar and the protection filter
- Empty and dispose of the Jar according to the laws in force in your country;
- Separate all parts of the cover (overflow valve, o-ring);

After disposing of disposable parts and disassembling the jar wash in running cold water and rinse thoroughly. Then soak in warm water (temperature shall not exceed 60°C). Wash thoroughly and if necessary use a non-abrasive brush to remove incrustations. Rinse in running warm water and dry all parts with a soft cloth (non-abrasive). The jar and the cover can be autoclaved by placing the parts into the autoclave and running one sterilization stem cycle at 121°C (1 bar relative pressure) making sure that the jar is positioned upsidedown.

After sterilization and cooling at environment temperature of the parts make sure that these are not damaged.

Assemble the jar as follows:

- Place the overflow valve into its seat in the cover (under VACUUM connector)
- Insert floating valve keeping the o-ring towards the opening of the cage
- Place the o-ring into its seat around the cover
- After completing assembling operations always make sue that cover seals perfectly to avoid vacuum leakages or liquid exit

The aspiration tubes can be sterilized on autoclave using a sterilization cycle at 120°C. The conical connector can be sterilized on autoclave using a sterilization cycle at 121°C. The device is ready for a new employment now.

**DO NOT WASH, STERILIZE OR PUT IN AUTOCLAVE THE ANTIBACTERIAL FILTER**

### 5.2 MAINTENANCE

#### 5.2.1 Precautionary maintenance

The person who carries out the precautionary maintenance of the appliance (user in person, manufacturer/supplier or a third party) has to guarantee the following basic requirements:

- Technical knowledge of the appliance and of the periodic maintenance procedures as described in these instructions.
- Specific qualifications and training in the maintenance operations of the appliance in question.
- The use of components/replacement parts/accessories that are either original or approved by the supplier, in such a way that each operation causes no alteration or modification to the appliance.
- Ability and skills to carry out control and maintenance operations on the device
- Guarantee complete adherence to the instructions of the Directive 93/42/CEE which includes also the obligation towards the manufacturer to maintain post sales records and traceability of the appliance if requested.

The device must undergo complete servicing every year by the manufacturer or by an authorized service centre.

The person responsible for every day maintenance can only substitute the spare parts indicated on paragraph 6.2 "Spare Parts".

All other substitutions or repairs can be carried out only by the manufacturer or by a centre authorised by the manufacturer.

#### 5.2.2 Special servicing

**Only the manufacturer or centres with written authorisation are authorised to complete any special servicing operations.**

The device, if used as indicated in the user manual, has an average life span of 5 years. The life span can be expanded only following a general revision of the product that must be carried out by the manufacturer or by a centre authorised by the manufacturer.

## 6 ACCESSORIES AND SPARE PARTS

|          |                                              |
|----------|----------------------------------------------|
| DG00150A | YANKAUER SUCTION TUBE                        |
| DG00212C | SUCTION CATHETERS                            |
| DG00214C |                                              |
| DG00216C |                                              |
| DG00218C |                                              |
| RV71104A | RVX 04 TRANSPORT BAG RES Q VAC IN ORANGE PVC |

### 6.2 SPARE PARTS

|          |                                                               |
|----------|---------------------------------------------------------------|
| RV71106C | ANTIBATERICAL FILTER                                          |
| RIRV004A | SPARE CANISTER FOR RES-Q-VAC                                  |
| OX10103A | SILICONE SUCTION TUBE FOR SUCTION DEVICE Ø 5x9 mm (PER METER) |
| SC70088A | DOUBLE SUCTION CONE                                           |

#### Warning

The information contained in this document could be modified without any warning and is not to be intended as a commitment on behalf of Spencer Italia S.r.l. Spencer products are exported to many countries and the same identical regulations are not always valid. For this reason there could be differences between the description here described and the product actually delivered. Spencer continually strives to reach the perfection of all items sold. We therefore hope you will understand if we reserve the right, at any time, to modify the shape, equipment, lay-out or technical aspects that are herein described.

© Copyright Spencer Italia S.r.l.

All rights reserved. No part of this document can be photocopied, reproduced or translated into another language without the written approval of Spencer Italia S.r.l.

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rédigé par Spencer Italia S.r.l.